TEST SYSTEM FOR DAILY VALIDATION
SEALING TEST (SEAL CONTROL) SHEET
 GMSEAL-CHECK Normal and GMSEAL-CHECK Tyvek®
 Operational Qualification for Testing of the Sealing
 Quality of Sealers (OQ)
 Daily Sealing (Validation) Test

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Order number	Product	Description / Content
222-110-060	GM SEAL-CHECK: Standard	Standard Daily Sealing Test Sheets According to EN-ISO 11607-2 & DIN 58953 Content: 250 PCs. into 1 package
222-110-070	GM SEAL-CHECK: Tyvek®	Tyvek® Daily Sealing Test Sheets According to EN-ISO 11607-2 & DIN 58953 Content: 100 PCs. into 1 package

Tested to comply with World Standard EN ISO 11607 part 2 and DIN 58953.

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222-110-060 GMSEAL-CHECK: *Standard*

222-110-070 GM SEAL-CHECK: *Tyvek®*

GMSEAL-CHECK-Standard and GMSEAL-CHECK- Tyvek® tests are designed as two models and should be selected according to the type of pouch you use. These tests should be carried out on a daily basis each morning before the sealer is used. These tests are described in EN-ISO 11607-2 and DIN 58953.

Remove the Seal-Check sheet from the package and place it in a bag of the appropriate width as shown in the figure below. Insert the bag into your device as usual. When the bonding process is completed, it will be seen clearly as if the following is successful or if there are any faulty areas. If you have received a defective device, contact your place of purchase to have it replaced with the required parts to restore your device to the factory settings..

Successful test results indicate that your device is in compliance with EN ISO 11607-2. The following test result can be documented daily.

If you have other technical questions on the subject, you can contact us. We are happy to give you technical information.

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